

**If you
use these...**



**Protect your nurses
from needlesticks.
Use UltraSafe[®]
Needle Guards.**



**Introducing
UltraSafe
Needle Guards**

UltraSafe Needle Guards are easily attached to prefilled glass syringes used with low molecular weight heparins, vaccines and biotechnology medicines. UltraSafe Needle Guards —protection that works. Just call us and we will take care of the rest. Call +1.760.918.9908.

**Get safe. Get UltraSafe
Anti-Needlestick Devices.**

UltraSafe Needle Guards


- Color-coded guards to assist in choosing correct size.
- High-strength, transparent plastic helps in medication identification.
- “Click” lock provides tactile and audible confirmation of guard activation.
- Familiar sliding guard activation and ease of assembly assist with compliance and user education.



Ultrasafe Needle Guard/Medication Cross Reference


B50

For use only with ISO standard 0.5ml prefilled glass syringes with a needle length of 5/8” or shorter.

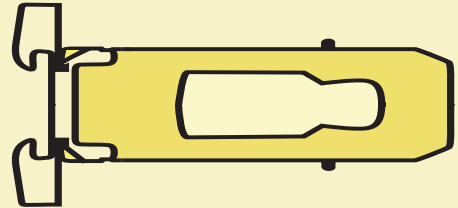


B100

For use only with ISO standard 1.0ml prefilled glass syringes with a needle length of 5/8” or shorter.

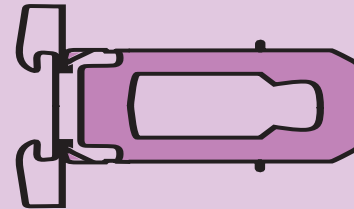


B100L

For use only with ISO standard 1.0ml (long) prefilled glass syringes with a needle length of 5/8” or shorter.

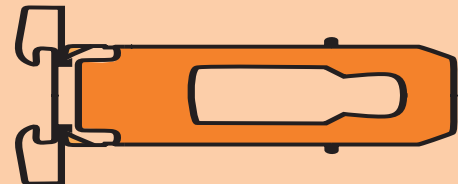


B150

For use only with ISO standard 1.5ml prefilled glass syringes with a needle length of 1” or shorter.

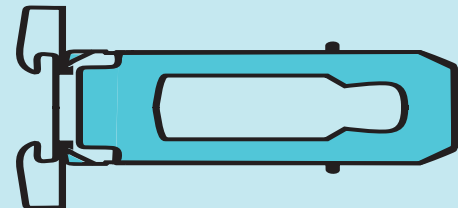


Directions For Use

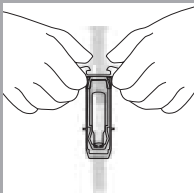


Insert and snap.

Insert syringe into needle guard until Finger grip snaps securely into place. Do not push plunger during assembly.

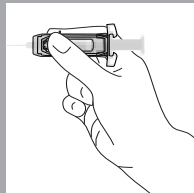

Administer injection.

Administer injection per standard protocol.

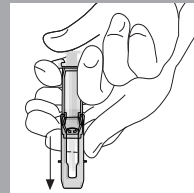

Slide safety guard over needle.

After injection, use free hand to slide safety guard over needle. An audible “click” indicates proper activation. Keep hands behind needle at all times.

Dispose.

Dispose of syringe/needle guard in approved sharps container.

